# MEDISANA®

## CardioDock® 2

**DE Blutdruck-Messmodul**

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[www.bluepoint-service.ch](http://www.bluepoint-service.ch)

- 1 Connection slot on your iPhone®, iPad® or iPod touch®
- 2 CardioDock blood pressure module
- 3 Upper arm pressure cuff
- 4 Air hose
- 5 CardioDock (rear view)
- 6 Connector for the cuff
- 7 Connection for micro USB plug
- 8 Start / Stop button 
- 9 Connector for your iPhone®, iPad® or iPod touch®

This instruction manual belongs to this device. It contains important information about starting up and operation. Read the instruction manual thoroughly. Non-observance of these instructions can result in serious injury or damage to the device.

**WARNING**

These warning notes must be observed to prevent any injury to the user.

**CAUTION**

These notes must be observed to prevent any damage to the device.

**NOTE**

These notes give you useful additional information on the installation and operation.

Classification: Type BF

Lot number

Manufacturer

Date of manufacture

**IMPORTANT INFORMATION!  
RETAIN FOR FUTURE USE!**

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

### Safety Information

#### 2.1 Tips for your well being

##### Proper use

- The system is intended for measuring the blood pressure from the upper arm of an adult. It should be used together with Apple (iOS 5.0 or higher) devices (iPhone 3GS, 4, 4S / iPod touch 3<sup>rd</sup> and 4<sup>th</sup> generation / iPad 1, 2 and 3).

##### Contraindications

- The system is not suitable for measuring the blood pressure of children. Ask your doctor before using it on older children.
- The device is not suitable for persons with acute arrhythmia.
- Irregularity of pulse or arrhythmia can lead to difficulties in recording a correct reading when measurements are taken using oscillometric blood pressure devices. This device is electronically equipped to detect arrhythmia and what are known as movement artefacts, which it indicates with the icon **ARR**. In this case, the reading may be imprecise. Inform your doctor so that he or she can assess the situation.
- If you should suffer from conditions such as arterial occlusive disease, please consult your doctor before using the unit.
- The unit may not be used to check the heart rate of patients with a pacemaker.
- Pregnant patients should take their own personal state of health into consideration and take all necessary precautions before using the unit. If in doubt, consult your doctor.
- Should you experience discomfort or other complaints during measurement, for example pain in the upper arm, please take the following action: Press the Start/Stop button **8** on the CardioDock or the Stop button on the display to deflate the cuff immediately. Loosen the cuff and remove it from your arm. Please contact either your supplier or MEDISANA directly.

- Do not use the cuff on open wounds, in order to prevent the spread of infection.
- Patients with a weak pulse should raise their arm and open and close their hand around 10 times before taking a reading. This exercise optimizes the pulse and the measuring process.

**WARNING**

**Do not take any therapeutic measures based on your own measurements! Never change the quantity of medication prescribed by your doctor!**

## 2.2 Important considerations

- The CardioDock is only to be used for the specific purpose described in this instruction manual.
- The system is not intended for commercial use.
- Any misuse will void the warranty.
- On the rare occasion of a fault causing the cuff to remain fully inflated during measurement, open the cuff immediately.
- Do not operate the system in the vicinity of strong transmitters such as microwave and shortwave appliances. This may impair the correct functioning of the unit.
- To ensure CardioDock readings are accurate, keep at a minimum distance of about 30 cm from other wireless devices.
- This module is not designed to be used by persons (including children) with limited physical, sensory or mental abilities, or by persons with insufficient experience and/or knowledge, unless under observation by a person responsible for their safety, or unless they have been instructed in the use of the device.
- Children must be supervised to ensure that they do not play with the device.
- Do not use the module if it does not work properly, if it has been dropped or has fallen into water or become damaged.
- Do not use the module in damp environments.
- Protect the module from moisture. Should moisture enter the unit, stop using it immediately. In this case, please contact either your supplier or MEDISANA directly.

- If a fault occurs, do not try to repair the module yourself. Attempts to do so will void the warranty. Refer all servicing to authorized service personnel.
- Do not drop the module and protect it against impact and strong vibration.
- Always keep the module clean and store it in a safe place. Protect the module from direct sunlight to prolong its service life.
- For cleaning, follow the instructions in "Care and maintenance".

### 2.3 Information about the software

- Make sure that your Apple device is running on the latest operating system and the VitaDock app is updated regularly. For information about app and software updates, see the instructions for your Apple device. The app shows a window with a warning when you change settings.
- Before the changes come into effect, you have to confirm the warning message.
- If you receive a call or a text message during a reading, the reading is cancelled and the cuff is deflated for safety reasons. If you want to prevent your reading from being cancelled, we recommend switching your iPhone to flight mode before the reading.

### 3.1 Items supplied and packaging

Thank you for your confidence in us and congratulations!

Please check first of all that the device is complete and is not damaged in any way. In case of doubt, do not use the appliance and contact your supplier or your service centre.

*The following parts are included:*

- 1 **CardioDock** blood pressure module
- 1 Pre-formed upper arm cuff size M (22 - 32 cm)
- 1 USB cable for charging battery
- 1 Carrying case
- 1 Instruction manual

If you notice any transport damage during unpacking, please contact your supplier without delay.

**WARNING**

Please ensure that the polythene packing is kept away from the reach of children!  
Risk of suffocation!

### 3.2 Influencing and evaluating readings

- Measure your blood pressure several times, then record and compare the results. Do not draw any conclusions from a single reading.
- Your blood pressure readings should always be evaluated by a doctor who is also familiar with your personal medical history. Take regular readings and inform your doctor from time to time.
- When taking readings, remember that the daily values are influenced by several factors. Smoking, consumption of alcohol, drugs and physical exertion influence the measured values in various ways.
- Measure your blood pressure before meals.
- Before taking readings, allow yourself at least five minutes rest.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.

**WARNING**

Blood pressure that is too low represents just as great a health risk as blood pressure that is too high!  
Fits of dizziness may lead to dangerous situations arising (e.g. on stairs or in traffic)!

### 3.3 Charging the blood pressure module's battery

The **CardioDock** blood pressure module has an integrated lithium ion battery (3,7 V $\equiv$ , 500 mAh).

- a. For charging the **CardioDock** battery, connect the **CardioDock** unit with an iPhone or iPad charging unit via the supplied USB cable. When **CardioDock** unit and iPhone are connected, the state of charge is displayed in the VitaDock screen.

- b. When the start/stop button  8 of the **CardioDock** unit is highlighted in blue, the state of charge of the battery is over 20 %. Should the charge level drop below 20 %, the LED colour changes from blue to yellow. Please charge the battery now. Avoid letting the battery run down completely, as this shortens the battery's service life.
- c. During the charging process, the start button is highlighted in yellow, until the battery is fully charged. When the battery is completely charged, the LED shortly lights up in blue, before the unit automatically switches off.


If the LED of the start/stop button  8 is flashing alternately in yellow and blue, the battery is defective.

#### 4.1 Installing the VitaDock® app

**MEDISANA** is setting new standards in mobile health with **VitaDock**. To operate your new **CardioDock**-blood pressure module for iPod touch, iPad and iPhone, you have to install the free **VitaDock** app. You can download it from Apple's app store and then install it.

- To find the **VitaDock** app in the App Store, enter the search term "**VitaDock**".
- Once the software has been successfully installed, the new "**VitaDock**" icon  appears on your device. Tap on the symbol to start **VitaDock**.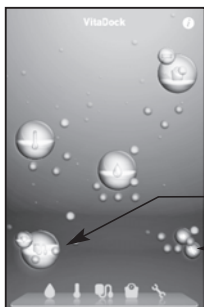

Welcome screen "**VitaDock** universe".

Symbol "CardioDock" 

Symbol "Settings" 

## 4.2 Functions of the VitaDock® App

**Disclaimer:** When a VitaDock module is connected, the module's serial number appears in the mast-head.

New reading (page 10)

Result of the last reading:

Systolic and diastolic pressure (mmHg) and pulse/min.  
Display of the readings as per WHO standard.

CardioDock statistics: Tap on the symbol "Stats" to display the statistics. The results of the last days will be displayed. Rotate your device by 90° and touch the relevant field to view additional test results.

CardioDock diary: Tap the arrows to page through the log book.

3MAM measurement: The 3MAM measurement can give you more precise readings, particularly if your blood pressure fluctuates. Three readings are taken one after the other, and a weighted average is calculated from this. There is a pause of 15 seconds between each reading. The display shows the time remaining until the next reading. The follow-on measurements start automatically; remain sitting calmly during the process. When the last measurement is complete, the result is displayed: You can add comments and save it, as normal.


Under "Settings" you can choose between WHO (World Health Organization) standard and custom settings. If you choose custom settings, enter the target blood pressure values specified by your doctor:

- Systolic
  - Diastolic
  - Pulse
- Save your entries.

### 5.1 Putting on the upper arm pressure cuff

- First attach the free end of the cuff's air tube to the designated connector **6** on the unit.
- Slip the preformed cuff over the bare left upper arm.
- Position the air hose in the middle of your arm in line with your middle finger (*ill. 1 a*). The lower edge of the cuff should be 1 - 2 cm above the crease of the elbow (*ill. 1b*). Pull the cuff tight and close the Velcro fastener (*ill. 1 c*).
- Only position the cuff on the right arm if it cannot be used on the left arm. Always carry out measurements on the same arm.
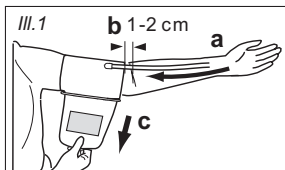


### 5.2 Correct position for taking readings

- If possible, take the reading while seated.
- Relax your arm and place it comfortably, on a table for instance. Keep still during the measurement: Do not move and do not talk as this may alter the measurements.

The correct measuring position when sitting is shown in *ill.2*.

- In special cases the measurement can also be taken when lying down. The correct measuring position when lying down is shown in *ill.3*.

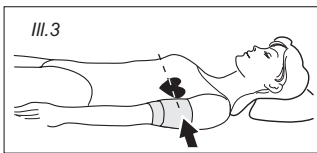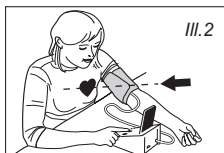The cuff should always be at the same height as your heart.

### 5.3 Taking blood pressure readings

To avoid disruption from an incoming call or text message during a reading, we recommend switching the iPhone to flight mode. You can activate the CardioDock's blood pressure measuring mode in two ways:

#### a) Plug & Play:

Put the Apple device, when switched on, onto the CardioDock module. The VitaDock app automatically starts in blood pressure measuring mode.

#### b) Manual start:

Start the VitaDock app. Tap on the symbol „CardioDock“  and on the symbol "New" . The „new measurement step 1" window opens. You are prompted to connect the CardioDock module (or enter the value manually). Place the Apple device on the CardioDock module.

Once you have put on the upper arm cuff and are in the measuring position, start the reading by touching **Start** or pressing the Start/Stop button  on the CardioDock module.

The module automatically pumps up the cuff. Remain at rest during the reading: Do not move and do not talk. While the cuff is pumping up, the pulse icon flashes. The rising pressure is shown as an increasing number, the pulse is shown as a wave.

#### Important:

- You can cancel the measurement at any time by touching **Stop** or pressing the Start/Stop button  on the module. The air escapes from the cuff.
- If a call or a text message is received during the reading, the measurement is cancelled. The cuff deflates.

### 5.4 The measurement result

The unit inflates the cuff until sufficient pressure has built up for a measurement. Next, the device lets the air out of the cuff and shows the reading:

- Systolic pressure
- Diastolic pressure
- Pulse rate

Touch the field **Add comment** to open the entry field for comments about the reading.

Optional comments:

- Activity (infirm  , rest  , normal activity  , sport  )

- Mood (  ,  ,  ) and notes (  )

Comments are displayed in the statistics and in the logbook.

### 5.5 Sending a report

Touch the "report" icon  in "global Settings" to display the monthly reports. Touch the arrows to reach the previous or next month. Slide the display to up or down to view the individual days in the month. Tap "Send report" to send the data by email, e.g. to your doctor. To send data by email, the email-function needs to be activated on your iOS-device.

### 5.6 Adding further user profiles

In "Personal settings" you have the option of creating up to 3 additional user profiles in the VitaDock app. Therefore log out of the current user profile by touching **Sign out**. You can then create an additional user profile on the following screen: **Add another user**.

 **The red highlighted input fields have to be filled in.**

### 5.7 Deleting user profiles

Close the actual user profile. Touch **Edit** and **Delete** to delete a user profile. The deleted user profile is shown in grey. For a period of 14 days, you have the option of reactivating the deleted user profile by touching it. After 14 days, the user profile is automatically deleted completely from the app.

#### **Caution!**

**After the 14 days, the user profile is irreversibly deleted. There is no way to restore it.**

## 6.1 Error messages

### Unusual reading

- Repeat the test.
- The cuff is not positioned correctly or the cuff is too small.
- You did not keep to the correct posture during the reading.
- During the reading, you spoke, moved your arm or body or you are excited or nervous.
- If you get another unusual reading, contact your doctor.

### Irregular pulse (arrhythmia)

- This device is not suitable for persons with an acute irregular pulse.

### The cuff does not pump up properly

- Check that the air hose is fitted correctly.
- The air hose is pinched or is leaking.
- The cuff is leaking.

### The device does not respond

- The Apple device is not connected properly to the blood pressure module. As appropriate remove the bumper.
- The battery is too weak.
- Separate the Apple device and the module, re-start the software and reconnect the Apple device to the module.
- The LED of the start/stop-button flashes alternately yellow and blue: The CardioDock battery is defective.

### 6.2 Care and maintenance

Your **CardioDock** blood pressure module is a high-precision instrument. Handle with care to avoid damaging the electronics and prevent defects.

*Observe the following instructions:*

- Never use strong detergents or hard brushes.
- Clean the unit using a soft cloth lightly moistened with a mild soapy solution. Do not allow any water to get into the unit. Do not use the unit again until it is completely dry.
- Do not expose the unit to direct sunlight, and protect it from dust and moisture.
- Only inflate the cuff when it is in position around the upper arm.
- Keep the cuff clean. If it gets dirty, separate it from the device and clean it with mild soapy water. Rinse it thoroughly with cold water and let it dry in the air. Do not put the cuff in a dryer and do not iron it.
- Even if you do not use the device for a long time, charge it fully once a month.
- Put the module back in its carrying case after each use.

#### *• Servicing and calibration*

The unit has been calibrated by the manufacturer for a period of two years' use. In the case of **commercial use**, the unit must be serviced and recalibrated at least once every two years. This calibration will be charged for and can be carried out by an appropriate authority or an authorised service centre - in accordance with the regulations for users of medical equipment.

### 6.3 Guidelines / Standards

This blood pressure monitor meets the requirements of the EU standard 93/42/EEC for non-invasive blood pressure monitors. It is certified in accordance with EC Guidelines and carries the CE symbol (conformity symbol) „CE 0483“.

**CE 0483**

The blood pressure monitor corresponds to European standards EN 60601-1/+A1/+A2, EN 60601-1-2, EN 1060-1/+A1, EN 1060-3/+A1.

#### Electromagnetic compatibility:

The device complies with the EN 60601-1-2 standard for electromagnetic compatibility. Enquire at **MEDISANA** for details on this measurement data.

### 6.4 Disposal

This module must not be disposed of together with domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Please remove the battery before disposing of the device / unit. Do not dispose of old batteries with your household waste, but at a battery collection station at a recycling site or in a shop. Consult your local authority or your supplier for information about disposal.

The packaging can be reused or recycled.

Please dispose properly of any packaging material no longer required.

### 6.5 Technical specifications

Product name	: <b>CardioDock</b> Blood pressure module
Runs on	: Apple devices: - iPhone 3GS, 4, 4S - iPod touch 3 <sup>rd</sup> and 4 <sup>th</sup> generation - iPad 1, 2 and 3
Operating system	: Apple iOS 5.0 or higher
Power supply	: via USB (5 V $\equiv$ , 500mAh) or integrated battery (3,7 V $\equiv$ , 500 mAh)

Measuring technique	: Oscillometric, Single readings or 3MAM readings
Pressure generation	: Automatic, using micro pump
Deflation	: Automatic
Blood pressure measuring range	: 30 – 280 mmHg
Pulse measuring range	: 40 – 200 beats / minute
Maximum error tolerance for static pressure	: $\pm 3$ mmHg
Maximum error tolerance for pulse rate	: $\pm 5\%$
Memory	: depends on memory of the connected Apple iPhone, iPad or iPod touch device
Operating conditions	: +10°C to +40°C, 15 % to 90% humidity
Storage conditions	: -20°C to +55°C, 15 % to 90% humidity
Dimensions ( $\varnothing$ x H)	: approx. 120 mm x 93,5 mm
Cuff size	: 22 – 32 cm adult cuff for average upper arm circumference (size M)
Weight	: approx. 213 g
Article number	: 51285
EAN number	: 40 15588 51285 8
Spare cuffs	: - Normal / medium (size M), 22 – 32 cm 51283 / 4015588 51283 4  - Large (size L), 30 - 42 cm 51284 / 4015588 51284 1

**In accordance with our policy of continual product improvement, we reserve the right to make technical and visual changes without notice.**

## 7.1 Warranty and repair terms

Please contact your dealer or the service centre in case of a claim under the warranty. If you have to return the unit, please enclose a copy of your receipt and state what the defect is.

The following warranty terms apply:

1. The warranty period for **MEDISANA** products is three years from date of purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Defects in material or workmanship will be removed free of charge within the warranty period.
3. Repairs under warranty do not extend the warranty period either for the unit or for the replacement parts.
4. The following is excluded under the warranty:
  - a. All damage which has arisen due to improper treatment, e.g. non-observance of the user instructions.
  - b. All damage which is due to repairs or tampering by the customer or unauthorised third parties.
  - c. Damage which has arisen during transport from the manufacturer to the consumer or during transport to the service centre.
  - d. Accessories which are subject to normal wear and tear (cuffs, batteries etc.).
5. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

### **MEDISANA AG**

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41468 NEUSS

Germany

E-mail: [info@medisana.de](mailto:info@medisana.de)

Internet: [www.medisana.de](http://www.medisana.de)

*For service, accessories and spare parts, please contact:*

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