beddit

USER GUIDE

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beddit

USER GUIDE	5
BENUTZERHANDBUCH	14
GUIDE DE L'UTILISATEUR.....	22
GUIDA PER L'UTENTE.....	30
MANUAL DEL USUARIO	38

Mobile device compatibility

Made for

iPhone 5s

iPhone 5c

iPhone 5

iPhone 4s

iPhone 4

iPod touch (5th generation)

Also compatible with selected Android devices.

See the full list at support.beddit.com


BEDDIT APP

WIRELESS SYNC

WHAT THIS PACKAGE CONTAINS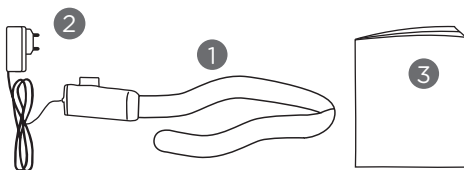

- 1 The Beddit sensor
- 2 Power adapter
- 3 User Guide

Beddit is not a medical device and should not be used to diagnose or treat medical conditions. Consult your doctor for prevention, diagnosis and treatment.

Beddit® is a registered trademark of Beddit Ltd.

iPhone and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries. All other trademarks are the property of their respective owners.

Patented, patents pending.
Designed and engineered in Finland.
Made in China.


Beddit Ltd., © 2014.

QUICK GUIDE

Follow these steps to install the Beddit sensor in your bed.


1 STRAIGHTEN THE SENSOR

Due to the packaging of the sensor, you may need to prepare the sensor for use. Place the sensor on a hard surface to straighten the sensor.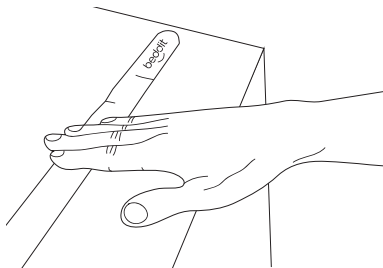


2 INSTALL THE SENSOR TO YOUR BED

The sensor film has an adhesive tape on one side. Remove the liner and attach the sensor like a sticker to your mattress or mattress cover under the bed sheets. Plug the sensor into the low voltage power adapter provided in this package.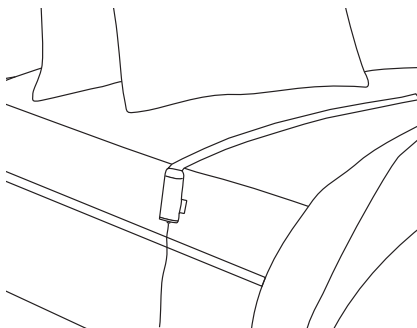

3 SENSOR IN BED


EN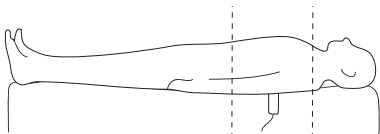


Place the sensor under your chest.

If you sleep alone in the bed, place the sensor so that the sensor strip extends fully (about 26 in, 65 cm) on the mattress.

If you sleep alone in a wide bed, place the sensor to cover the area where you normally sleep. If two people sleep in the bed, place the sensor so that there is a 6-inch gap (15 cm) between the midline of the bed and the sensor's end. With beds over 63-inch wide (160 cm), the gap may be wider.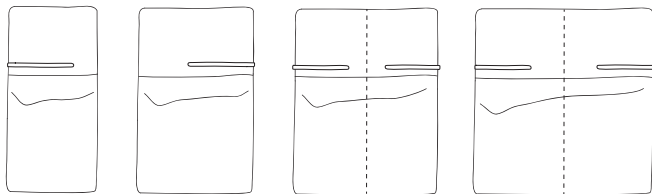

DOWNLOAD THE BEDDIT APP

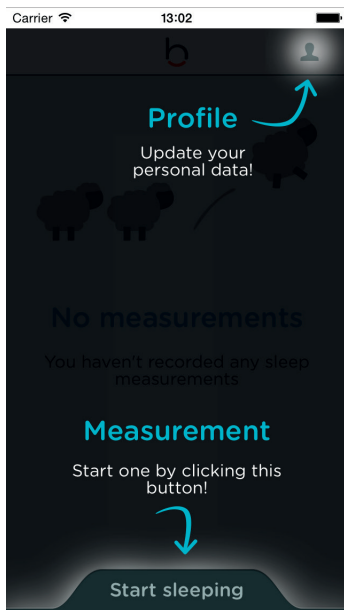Download the Beddit app for your mobile device from App Store or Google Play.

Launch the application and follow the on-screen instructions.

CARE INSTRUCTIONS

Instructions for how to properly take care of your Beddit and maximize its life.

1. Clean the top side of the sensor with a damp cloth if needed.
2. Do not wash the sensor in a washing machine or dry it in a dryer. Doing so will ruin it and void your warranty.
3. The tape that holds the sensor to your mattress may lose its adhesiveness over time. This is normal and you can easily replace the tape. See this user guide for instructions.
4. Do not try to repair the sensor, cable or power adapter other than replacing the adhesive tape of the sensor. There are no parts that can be safely repaired at home. Contact us if you need help with your Beddit. If there is a defect covered by the warranty, we will replace the defective part or refund your money. See the warranty terms and limitations.

5. When making your bed, be careful with the sensor and the power cable to avoid damaging them. If you have an adjustable mattress or rollaway bed, do not fold the sensor and cable into the bed. Do not allow the sensor or cable to be folded up or tangled in your bedding.

6. Your Beddit includes a power adapter. Use it for powering the sensor. Using the USB port on a laptop or other electronics could result in power variation or damage to the sensor or your electronics or data. Plugging the power cable into your electronics is at your own risk.


Visit support.beddit.com for latest updates and information.

HOW TO REPLACE THE ADHESIVE TAPE

The adhesive tape on the sensor strip may lose its grip if the sensor is re-attached many times. The tape can be easily replaced. Grab the sensor strip with one hand at the sensor end and work with the other hand to remove the tape. Carefully peel off the old adhesive layer from the sensor. Replace with a thin double-sided tape.

When peeling off the adhesive tape, do not grab the sensor module or cable as that may cause damage.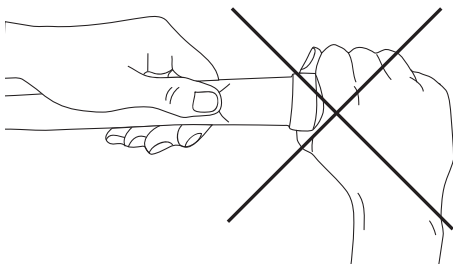

IMPORTANT SAFETY INFORMATION

Read and understand these warnings. Failure to follow them could result in death or serious injury.

1. Beddit is not a medical device. It is not designed or intended to diagnose or treat any medical conditions, and you should not use it for these purposes. Consult your doctor for prevention, diagnosis, and treatment of sleep disorders and other medical issues.

2. Be careful when placing the sensor cable to avoid trip hazards.

3. Do not use Beddit with infants or children. Death or injury could result from the sensor or power cable getting around an infant's or child's neck during their sleep.

4. Similarly, if your sleeping habits put you at risk of entangling a bed-side cord or an under-sheet sensor around your neck while sleeping, do not use Beddit.

5. Beddit is not a baby monitor. It will not warn you of any sleeping problems your infant may have. It will not warn you if your infant stops breathing. It will not warn of or prevent sudden infant death syndrome.

6. Do not use Beddit around water or other liquids. Like any electrical device, there is an electrocution hazard if you get Beddit wet while it is plugged in.

7. Do not disassemble the sensor or power adapter. There are no repairs that you can safely do on them other than replacing the adhesive tape of the sensor. See use and care instructions on how to replace the adhesive tape.

EN

LIMITED WARRANTY

This Limited Warranty is applicable globally except in the United States of America. Your Beddit consists of a Bluetooth-enabled sensor, a USB power adapter and a mobile app, which you must download separately. This Limited Warranty covers only the sensor and power adapter. You have legal rights under applicable national legislation governing the sale of consumer goods. Those mandatory rights are not affected by this Limited Warranty. See translations of the warranty terms in your language and the warranty terms for the United States of America at www.beddit.com

WHAT IS COVERED

Beddit Ltd. warrants the sensor and power adapter against defects in materials and workmanship under normal use for one year from the date of original purchase by the end user.

This Limited Warranty is applicable globally except in the United States of America.

WHAT IS NOT COVERED

This Limited Warranty does not apply to normal wear and tear or damage caused by any of the following: misuse (including failure to follow product documentation), neglect, disassembly, washing in the washing machine or drying in the dryer, pets, water or liquid spills, alterations, unauthorised repairs or external causes such as thermal or environmental conditions. It also does not cover loss of the adhesive from repeated applications of the sensor to bedding (see product instructions on how to replace the adhesive tape). Beddit Ltd. does not provide any warranty for the mobile app or the device that runs the app. This Limited Warranty does not guarantee

that the use of your Beddit with the mobile app will be uninterrupted or error free.

CONTENTS OF THE LIMITED WARRANTY

If a defect arises and you return the product during the one-year warranty period, we will in our sole discretion and to the extent permitted by law either replace the defective part with a new or refurbished one that is functionally at least equivalent to yours, or refund all or part of the amount you paid. If we replace the product, the replaced product will continue to be warranted for the remaining time of the original warranty period. All returned parts for which you have received a refund or replacement will become the property of Beddit Ltd.

To the maximum extent permitted by applicable law, Beddit Ltd. shall not be liable to you for any indirect or consequential damages that may be incurred by you in connection with Beddit. To the maximum extent permitted by applicable law, Beddit Ltd.'s total liability in connection with Beddit or this Limited Warranty will not exceed the purchase price that you actually paid for your Beddit.

REPAIRS

There are no repairs that you can safely do on the sensor or power adapter except replacing the adhesive tape of the sensor. The adhesive tape on the sensor will lose its stickiness over time as the sensor is attached to and removed from your bed. This is normal, and it is not covered by this Limited Warranty. See the product instructions for how to replace the adhesive tape. All other repairs are unauthorised and will void this Limited Warranty.

If you replace the adhesive tape, Beddit Ltd. is not liable for any damages to the sensor, your bed, your bedding or any property caused by the replacement adhesive tape.

HOW TO OBTAIN WARRANTY SERVICE

You must present the following information to make a warranty claim: (i) model of your product; (ii) your full address and contact information; and (iii) your purchase receipt or other proof of the date purchased.

To make a warranty claim, if you purchased the product directly from Beddit, please contact us at support@beddit.com for warranty service. If you purchased the product from an authorized dealer, contact the dealer where you purchased the product.

No employee or representative of Beddit Ltd. or any third party is authorised to modify, extend or add to this Limited Warranty.

In regard to the performed tests the EUT fulfils the requirements defined in the test specifications EN 301 489-1 V1.9.2, EN 301 489-17 V2.1.1, EN300 328 V1.7.1. Test Report: 242978B

Contains Module WT12-A-AISAP whose FCC Identifier is QOQWT12

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This Class B digital apparatus complies with Canadian ICES-003.

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EUROPEAN DECLARATION OF CONFORMITY

We, Beddit.Com Oy
Of Kimmeltie 3, 02110 ESPOO, Finland

declare under our sole responsibility that the product
Equipment BEDDIT, White/Black, EUROPE

to which this declaration relates is in conformity with the following standard(s) or other normative document(s)

EN 301 489-1 V1.9.2	ERM; ElectroMagnetic Compatibility standard for radio equipment and services; Part1: Common Technical Requirements
EN301 489-17 V2.1.1	ERM; ElectroMagnetic Compatibility standard for radio equipment and services; Part17: Specific Conditions for Broadband Data Transmission Systems
EN300 328 V1.7.1	ERM; Wideband transmission Systems; Data Transmission equipment operating in the 2.4GHz ISM band and using wide band modulation techniques; Harmonized EN covering essential requirements under article 3.2 of the R&TTE Directive

following the relevant provisions of the directive 2004/108/EC (EMC), 2006/95/EC (LVD), 2011/65/EU (RoHS).

The Technical Construction File is maintained at:
Company: Beddit.Com Oy
Company Address: Kimmeltie 3, 02110 ESPOO, Finland

The authorized representative located within the Community is:
Name: Lasse Leppäkorpi
Title: Chief Executive Officer

Date of issue: 21.02.2014
Place of issue: ESPOO; FINLAND

Legal Signature:

beddit

www.beddit.com
support@beddit.com